


EMERGENCY FACE SHIELD

1. PACKAGE CONTENTS

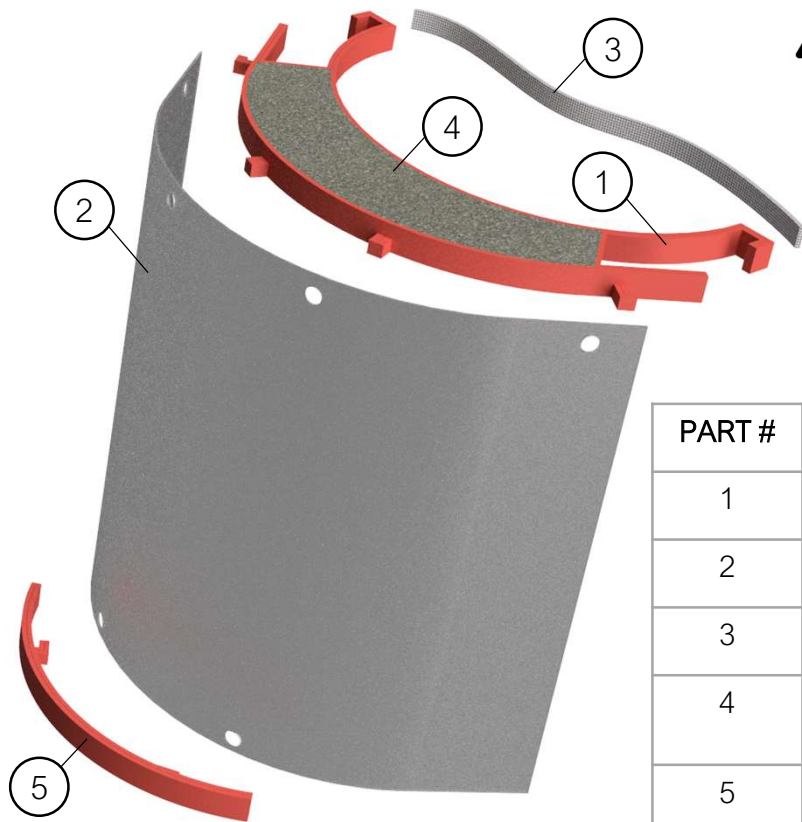

Please note, this face shield is currently a non-CE marked product. As such, it should only be treated as a secondary additional layer of protection or used in an emergency when no CE marked alternatives are available

PART #	NAME	QUANTITY
1	Headband	1
2	Visor	1
3	Strap	1
4	Foam Insert (<i>optional</i>)	1
5	Base Support	1

2. ASSEMBLY INSTRUCTIONS

Step 1: Attach strap (3) to headband (1)



Step 2: Insert foam insert (4) (*optional*) and check for fit to head. Trim if necessary

Step 3: Attach base support (5) to bottom of visor (2)

Step 4: Attach visor (2) to central studs of headband (1)

Step 5: Complete attachment of visor (2) to outer studs of headband (1)

Video available online at: <https://warwick.ac.uk/faceshield>

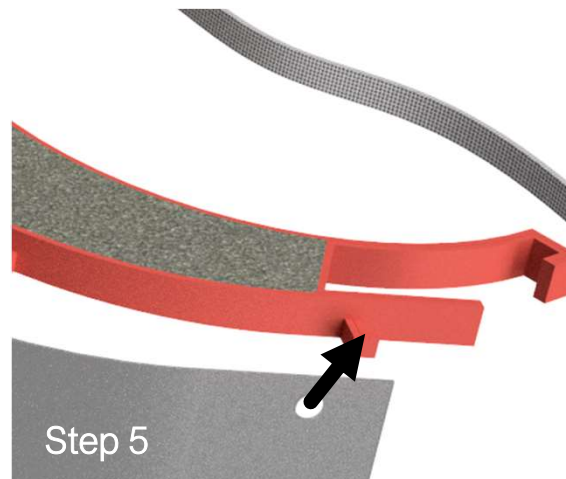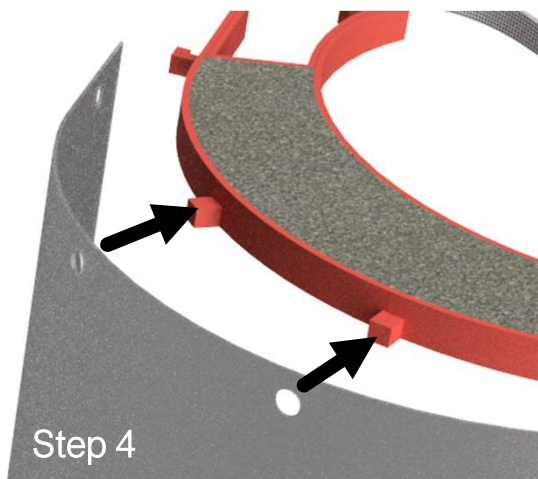

Email Contact: engbuildspace@warwick.ac.uk